



# HBOT EPE

## - Ear Pressure Equalizer -

The Maumee Scientific Ear Pressure Equalizer (EPE) is a reusable item designed for Hyperbaric Oxygen Therapy patients. It helps to overcome negative ear pressure as the chamber gets pressurized.

### INSTRUCTIONS FOR USE



The patient places the EPE in front of one of his/her nostrils. The other nostril is closed by applying pressure with one finger and exhaling through the nose, trying to inflate the balloon. This can be repeated several times until ear pressure is equalized.

The EPE is for use in hospitals or other clinical settings by individuals that have at least minimal instructions or training on the use of the EPE.

- REUSABLE Ear Pressure Equalizer to help patients with negative ear pressure problems.
- ONE SIZE to cover a wide range of patient nose morphology.
- SPECIFICALLY developed for Hyperbaric Oxygen Therapy patients.
- MANUFACTURED OUT OF STAINLESS STEEL to offer a unique robustness.
- EASY TO CLEAN (do not use chlorine type of detergent).

### TECHNICAL SPECIFICATIONS

DIMENSIONS	Hight: 37,8mm - Weight: 20 mm
WEIGHT	43 g
MATERIAL	Stainless Steel 304
PART NUMBER	MS.80.0002

